

# User seal checks

Half-face reusable respirators | filter

## NEGATIVE AND POSITIVE PRESSURE




**Cover filter inlets  
with thumbs**

Breathe **in** gently

Respirator should  
slightly **collapse**

If air leaks in  
around the face:

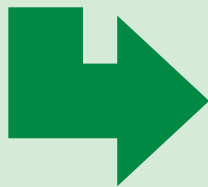

**Adjust straps or  
Reposition face piece**




**Cover exhalation  
valve with palm**

Breathe **out** gently

Respirator should  
slightly **expand**

If air leaks out  
around the face:

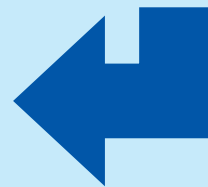

**REPEAT TEST UNTIL SEAL IS ACHIEVED**